



## Quickstart Guide: Step-by-Step™ with Levels communicator

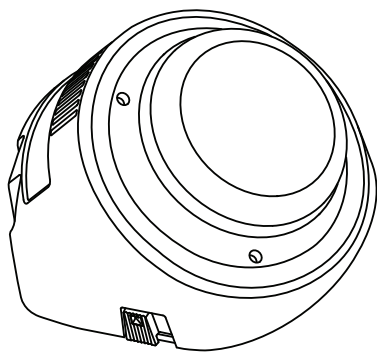

**ablenet**

www.ablenetinc.com


### 1 Choose your Switch Color

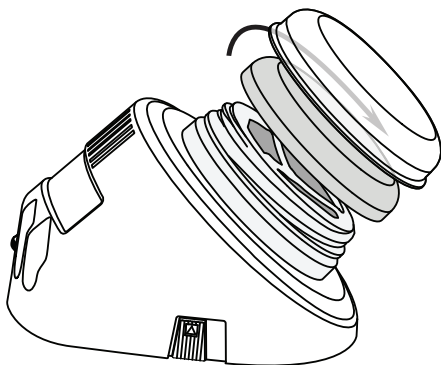

**Firmly, rotate the colored switch top counter-clockwise to remove. Attach a new color by rotating the cap clockwise until it stops.**

A Snap Switch Cap is included with your Step-by-Step with Levels. Place a picture or symbol on the switch top, and secure by snapping your Switch Cap over it.


### 2 Insert Batteries and Turn On

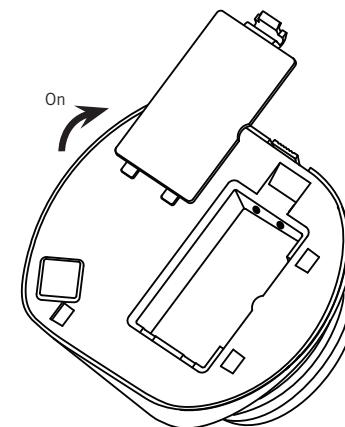

**Connect one 9-volt battery at the bottom of the device. Rotate the Volume ON/OFF knob to turn the device on.**

The Step-by-Step with Levels includes a detachable Mounting Base that must be removed to access the battery door. Press the tab and slide to remove the base.

### 3 Select Level & Enter Record Mode

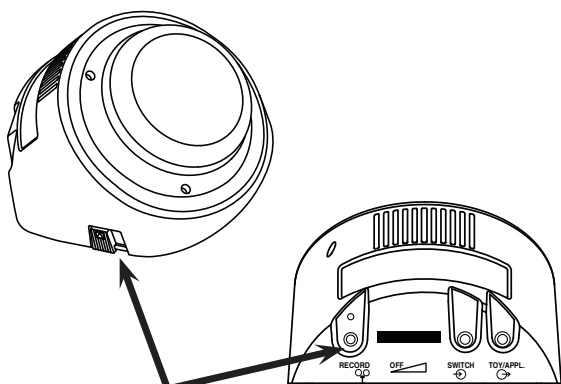


Press and Select

**Press and Hold the RECORD button until the RECORD Light activates, then Release.**

**Select desired Level using the Level switch on the side of the device.**

### 4 Record a Series of Messages

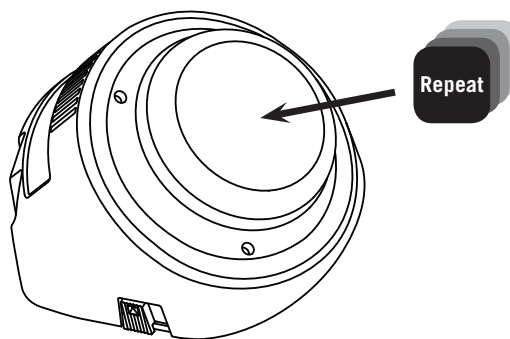


**Press and Hold the switch top, begin speaking when you hear a beep. Release when finished.**

**Repeat for additional messages in a sequence.**

**Switch Level and repeat above steps to record additional Levels.**

RECORD Light will flash while recording.

### 5 Exit Record Mode

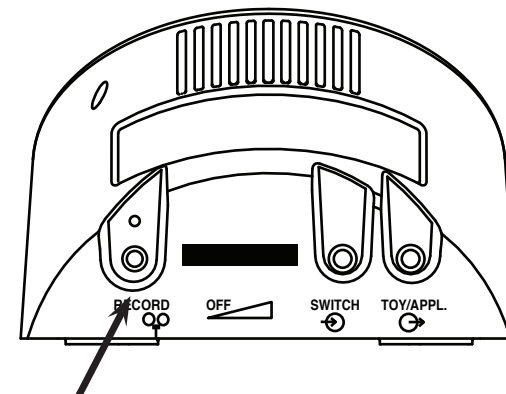


Press and Release

**Press and Release the RECORD button to exit Record Mode. Your Step-by-Step with Levels is ready for use.**

*NOTE: This device has 4 minutes of total recording time. The light will turn yellow and flash when only ten seconds of time remain.*

## Advanced Features

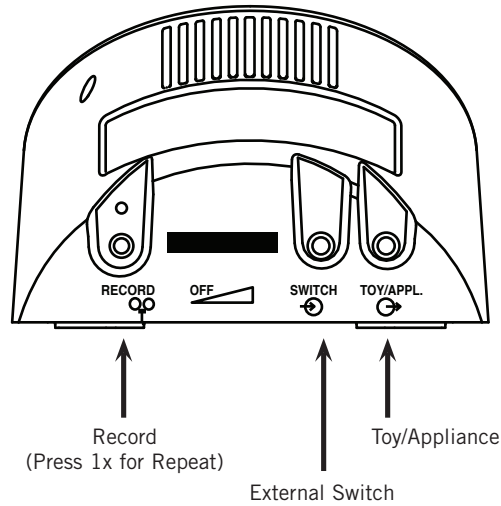

### Playing a message with an External Switch

Attach the switch of your choice with a 1/8-inch (3.5mm) plug into the switch input jack.

### Controlling Toys or Appliances



While recording a message, press the Record Button one time (1x) to assign a Toy/Appliance to that message. When that message is played, an attached toy or appliance will activate for the length of the recorded message.

*NOTE: To control an electrical appliance, a PowerLink® 3 is required. For information about adapting toys and appliances, visit [www.ablenetinc.com](http://www.ablenetinc.com)*

### Repeating a Message

1. Play the message you want repeated.
2. Press the Record Button one time after the message has ended. *The same message will repeat with each activation of the Switch Top*
3. To continue to the next message, press the Record Button one time (1x).

## Mounting Options

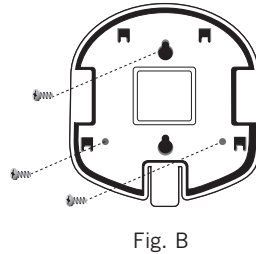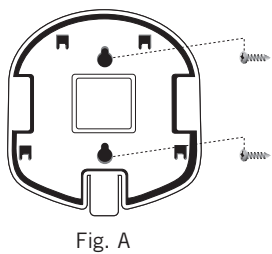

### Working with the Mounting Base

Release the base by pressing the quick release tab on the mounting base and sliding towards the back of the device.

Attach the base by aligning the four posts on the Base with the corresponding holes on Step-by-Step with Levels. Slide the base forward until it clicks.

For a temporary mount, use a #8 pan head screw and the two vertical slots on the Mounting Base. (Figure A)

For permanent mounting, screw the mounting base into place with the three holes shown at left. (Figure B)

### Additional Mounting Options (available at [www.ablenetinc.com](http://www.ablenetinc.com))



- UltraStik® or Dual Lock™ for flat or vertical surfaces.
- Travel Strap for waist or shoulder wear.
- Mounting Systems:
  - Universal Mounting System with a Knob or Lever
  - Slim Armstrong®
  - Sensitrac Flat Pad
  - Sensitrac Pad with Adjustable Arm

*Note: Specify large or small triangular mounting plate size when ordering mounting system.*

Slim Armstrong  
Mounting System



## Product Map and Warranty

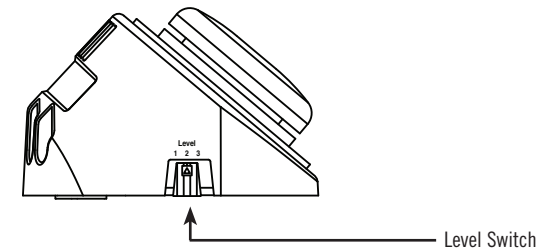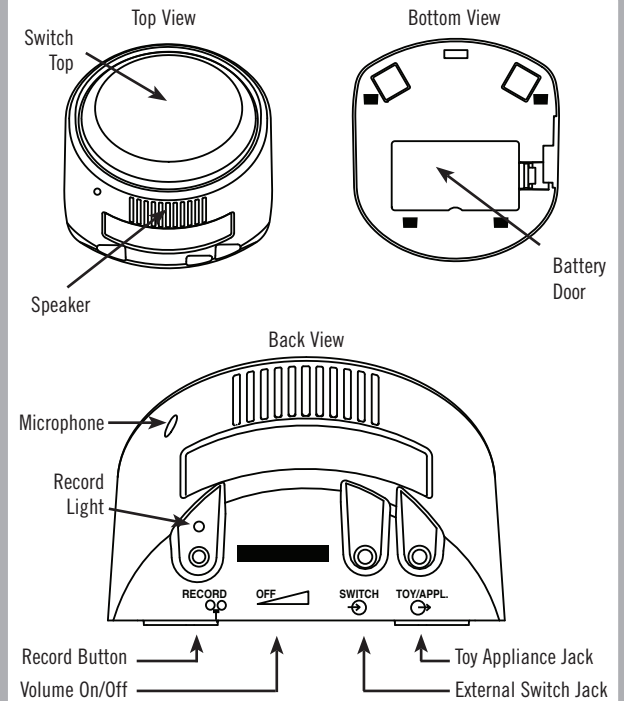

### Warranty/Repair/Return Information

AbleNet provides a limited one-year warranty on the Step-by-Step with Levels communicator product. For full warranty details, visit [www.ablenetinc.com](http://www.ablenetinc.com)

### For more information on AbleNet products and services:

AbleNet, Inc.,  
Minneapolis/St. Paul, MN  
800-322-0956 (US & Canada)  
Internet: [www.ablenetinc.com](http://www.ablenetinc.com)  
© 2007 AbleNet, Inc., All Rights Reserved