

Quickstart Guide:

Lingo® wearable messaging and communication aid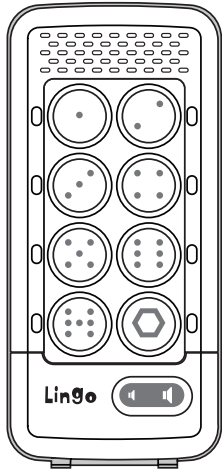

Overlays and ideas at www.ablenetinc.com

1 Charge and Turn On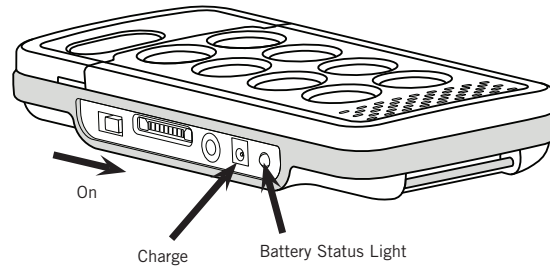


Plug the included wall adapter into the side of Lingo and charge until the Battery Status Light turns GREEN.

NOTE: When charging from very low charge (Flashing RED), three hours are required for a full charge.

NOTE: When Lingo needs to be charged, the Battery Status Light will flash Red three times.


2 Enter Record Mode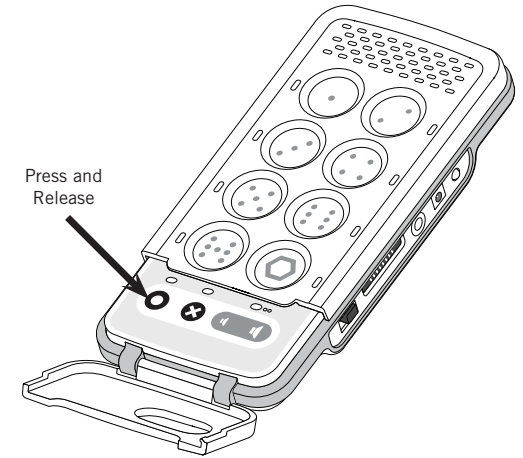

Press and Release the Record button. The Record Light activates.

NOTE: The Message Status Lights will also indicate their availability. Red = Recorded, Yellow = Blank

3 Record Single Messages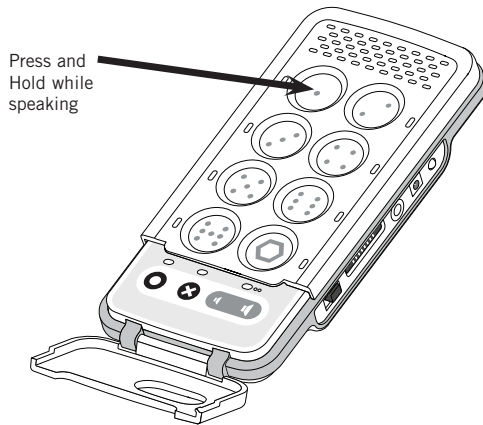


Press and Hold the message location, begin speaking when you hear a beep. Release when finished.

Repeat for additional message locations.
(Up to 25 seconds for locations 1-7.)

NOTE: Record Light will flash Red while recording, Yellow when only ten seconds remain.

4 Record Sequential Messages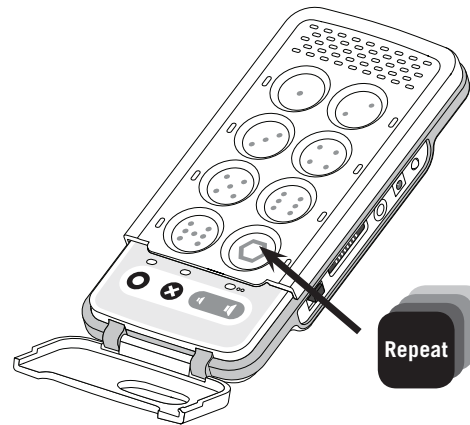


Repeat Step 3 for sequential messages in the location shown.

(Up to 60 seconds for this location.)

NOTE: Record Light will flash Red while recording, Yellow when only ten seconds remain.

5 Exit Record Mode

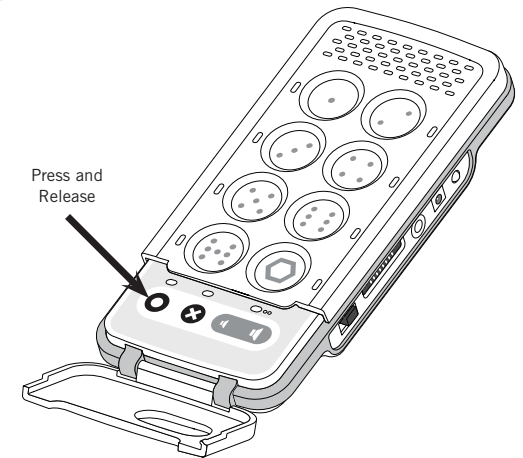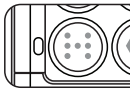
Press and Release the Record button to exit Record Mode. Your Lingo is ready for use.

NOTE: Repeat Steps 2, 3, and 5 with the Erase button to erase messages in any location.

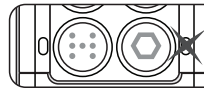
Inserting and Deleting Messages in a Sequence

The Sequential Message Location has the ability to play back a sequence of messages in order. To modify any single message in this location, use the instructions below.


To Insert a message within a sequence:

1. Press  until the message you wish to *insert after* has played. 
2. Press  2 times (2x) to enter Insert Mode. (The Record and Message lights will activate.) 
3. Press and hold  while speaking. Release when finished. (Lingo will beep and lights will activate while recording.) 
4. Lingo will exit Insert Mode when complete.


To Delete a message within a sequence:

1. Press  until the message you wish to *delete* has played. 
2. Press  2 times (2x) to enter Delete Mode. (The Erase and Message lights will activate.) 
3. Press and release . (Lingo will beep and lights will activate while deleting.) 
4. Lingo will exit Delete Mode when complete.

Accessories and Advanced Features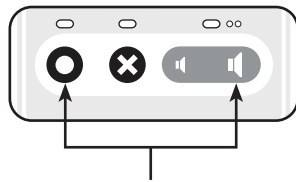

Press and Release to Lock/Unlock Control Panel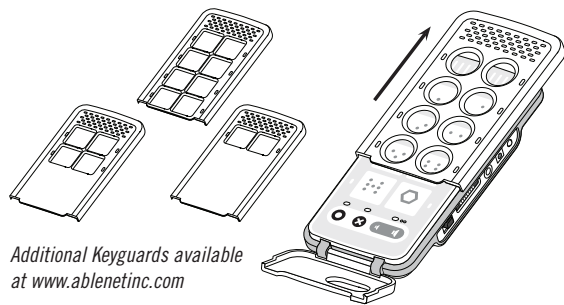

Additional Keyguards available at www.ablenetinc.com

Control Panel Lock-out

To lock or unlock the Control Panel, press the Record and Volume Up buttons simultaneously. When locked controls will not function.

*The Control Panel lights will indicate status:
Red = Locked, Yellow = Unlocked*

Master Clear

Press and Hold the Erase button for 10 seconds to erase all messages stored on Lingo.




Removing the Keyguard

To remove Lingo's Keyguard, slide the Keyguard up the surface of the device towards the speaker grill until free. Additional Keyguards sold separately.

Installing the Belt Clip

1. Attach Velcro® ends of the included strap to the Mounting Points on the back of Lingo.
2. Slide the Belt Clip onto the plastic knob, pressing until it clicks into place. (To Release, Press the tab on top of the Belt Clip.)

Product Map and Warranty

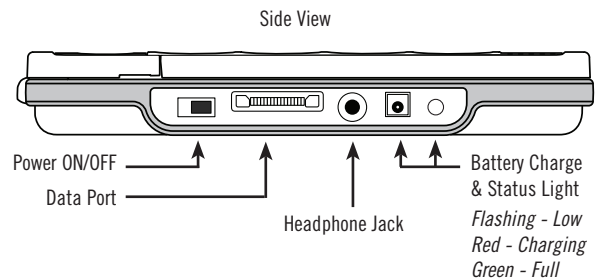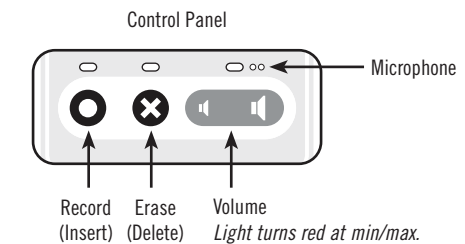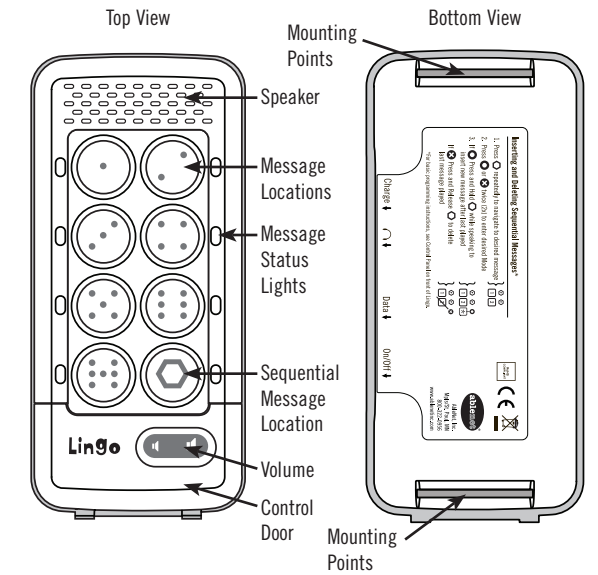Warranty/Repair/Return Information

AbleNet offers a limited one-year warranty on the Lingo product. For full warranty details, visit www.ablenetinc.com

For more information on AbleNet products and services:

AbleNet, Inc.,
Minneapolis/St. Paul, MN
800-322-0956 (US & Canada)
Internet: www.ablenetinc.com
© 2008 AbleNet, Inc., All Rights Reserved

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© 2008 AbleNet, Inc., Todos los derechos reservados