

Mount'n Mover QUICK GUIDE

READ SAFETY TIPS BEFORE USING

Maximum load = 15 pounds

Be sure mount is secure:


- Check bracket to wheelchair
- Check mount to post
- Check device to tilt plate

Do not attach to armrests

Do not unlock/move mount when wheelchair is tilted

Put in locked position before driving or tilting wheelchair


Stay clear of pinch points

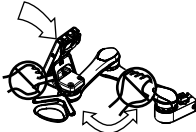

Adjust friction for specific user

For more info, read full manual
For support: 888.724.7002



PARTS

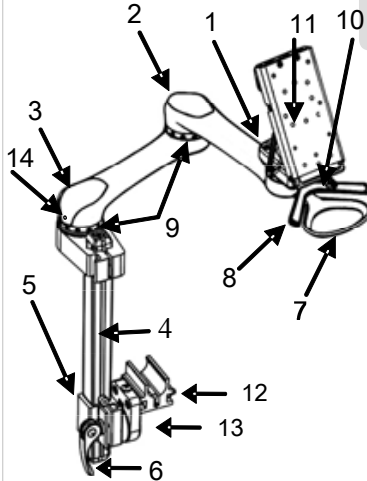


1. Wrist
2. Elbow
3. Shoulder
4. Post
5. Square Clamp
6. Lever
7. Hoop
8. Paddle
9. Lock Setters
10. Tilt Lock Lever
11. Quick Release Plate
12. Round Clamp
13. Wheelchair Bracket
14. Friction Adjustment Screw

POST ADJUSTMENTS

1. Unlock
2. Align ridge w/ gap

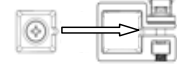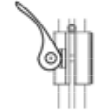

3. Insert post



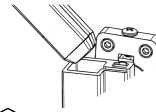


4. Lock



Height Setting



Loosen, adjust & tighten the height set screw

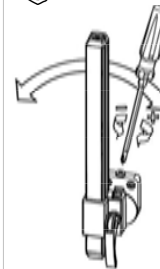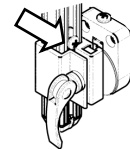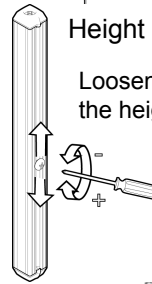


Setting post tilt angle: Loosen, adjust & tighten tilt screw.

QUICK RELEASE PLATE


Attaching and locking a quick release plate

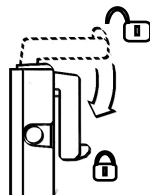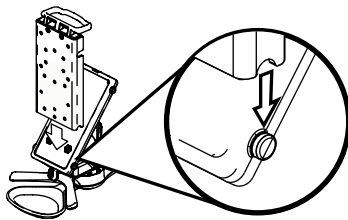

Tilt angle & lock adjusting the tilt

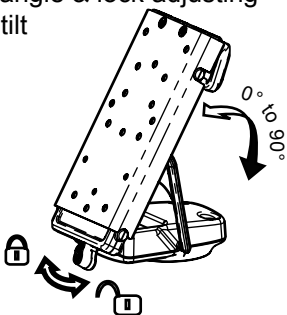


MOVING THE Mount'n Mover

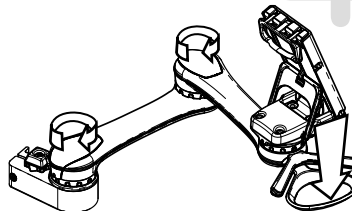

Hoop unlocks shoulder/elbow


Paddle unlocks wrist

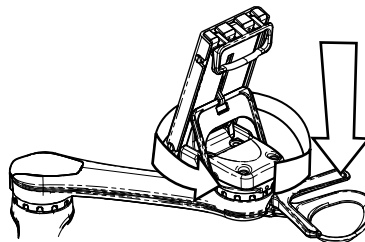


For more info:
Refer to full manual
Contact BlueSky Designs
1.888.724.7002
www.blueskydesigns.us

SETTING LOCK POSITIONS

Lock setters provide repeatable user-specific positions.

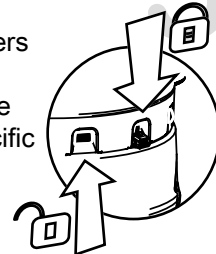


Lock position:

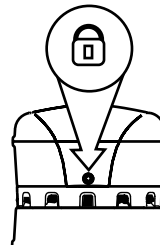

Lock setter is down and in-line with friction adjustment screw

Bypass position:

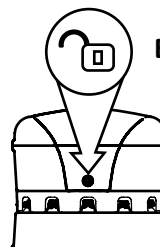

Lock setter is up